


Operation Instructions for the MDA Power Lifter

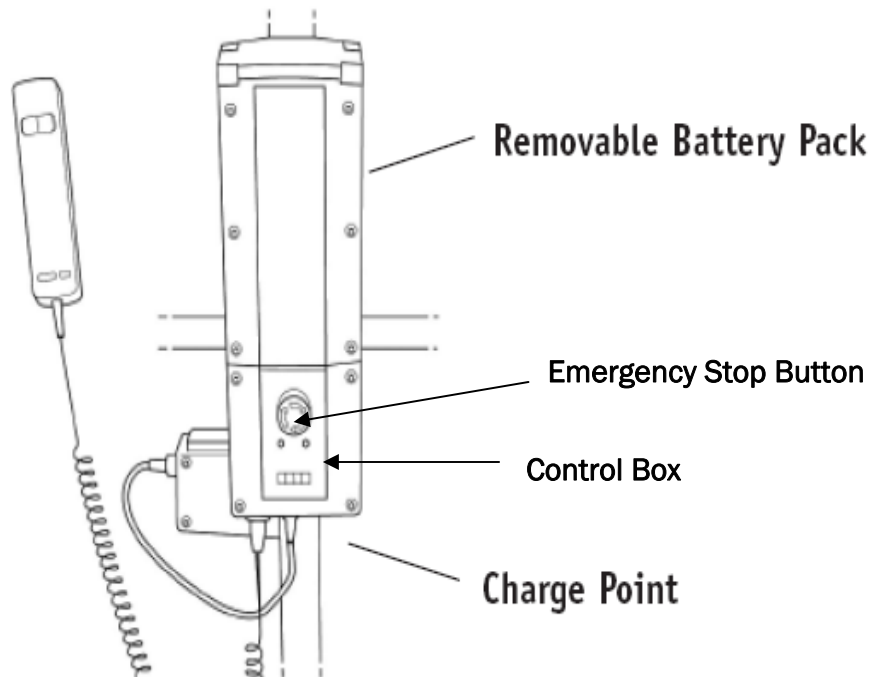

Hand Control: To activate the lifter use the “up” and >“down” arrows to move in the required direction. The lifter will not work if the emergency stop button is pushed in and/or the lifter is plugged into a wall socket.

Battery Charging: Plug the charger in the wall socket. Recharging the batteries takes approximately 6 hours. When the “Charge” light goes out on the control box that means the battery is fully charged. The battery alarm will sound when the batteries are fully charged.

Continuous Operation: The lifter should not exceed six minutes of continuous operation. After six minutes, the lifter will stop operating. If this occurs, turn off the lifter, wait a few minutes and then restart. It is important this does not occur as it can lead to other malfunctions with the lifter.

Emergency Stop Button: The emergency stop button located on the control box should be pushed in when the lifter is not in use for an extended period of time.



Manual Emergency Release Ring: If the lifter is stuck in the up position, pull the manual emergency release ring located on the back of the actuator.

Please Note:

- Do not charge the battery while a patient is in the lifter; especially if the unit is in the bath or wet areas.
- The base should be widened for stability.
- Never block the wheels when lifting a patient.
- Check that all nuts and bolts are secure before every use.
- Ensure sling attachments are not worn before operating the lifter.